Lancy

Restylane®
EYELIGHT™

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1. Data on file (MA-46075). European market survey by Opticom and Galderma Q4, 2020.
2. Stutman RL, Codner MA. Tear Trough Deformity: Review of Anatomy and Treatment Options. *Aesthetic Surgery Journal*. 2012 May;132(4):426-40.
3. Goldberg RA, Fiaschetti D. Filling the Periorbital Hollows With Hyaluronic Acid Gel: Initial Experience With 244 Injections. *Ophthalmic Plastic & Reconstructive Surgery*. 2006 Sep;22(5):335-41.
4. Kranendonk S, Obagi S. Autologous fat transfer for periorbital rejuvenation: indications, technique, and complications. *Dermatol Surg*. 2007 May;33(5):572-8.
5. Sadick NS, Bosniak SL, Cantisano-Zilkha M, Glavas IP, Roy D. Definition of the tear trough and the tear trough rating scale: Tear trough rating scale. *Journal of Cosmetic Dermatology*. 2007 Nov 30;6(4):218-22.
6. Goldberg RA, McCann JD, Fiaschetti D, Simon GJB. What Causes Eyelid Bags? Analysis of 114 Consecutive Patients: Plastic and Reconstructive Surgery. 2005 Apr;115(5):1395-402.
7. Lee J-H, Hong G. Definitions of groove and hollowness of the infraorbital region and clinical treatment using soft-tissue filler. *Arch Plast Surg*. 2018 May;45(3):214-21.
8. Verpaele A, Strand A. Restylane SubQ, a non-animal stabilized hyaluronic acid gel for soft tissue augmentation of the mid- and lower face. *Aesthetic Surgery Journal*. 2006 Jan;26(1):S10-7.
9. Data on file- data submitted for publication (NEW study).
10. Data on file- data submitted for publication (Consensus).
11. Data on file (MA-39680). Global 50 Million Restylane Treatments, 2021 (Restylane 25 years valid from September 30, 2021).
12. Data on file (MA-33947). Product integration comparison of different hyaluronan based fillers after injection in human ex vivo skin (2017).
13. Narins RS, Dayan SH, Brandt FS, Baldwin EK. Persistence and Improvement of Nasolabial Fold Correction with Nonanimal-Stabilized Hyaluronic Acid 100,000 Gel Particles/mL Filler on Two Retreatment Schedules: Results up to 18 Months on Two Retreatment Schedules. *Dermatol Surg*. 2008 Jun;34(s1):S2-8.
14. Andriopoulos B et al. Poster presented at AMWC 2019.
15. Data on file (MA-43049).
16. Lundgren B, Sandqvist U, Bordier N, Gauthier B. Using a New Photo Scale to Compare Product Integration of Different Hyaluronan-Based Fillers After Injection in Human Ex Vivo Skin. *J Drugs Dermatol*. 2018 Sep 1;17(9):982-6.
17. Data on file (MA-47865). Summary of Restylane Publications Analysis.
18. Molina B, David M, Jain R, Amselem M, Ruiz-Rodriguez R, Ma MY, et al. Patient Satisfaction and Efficacy of Full-Facial Rejuvenation Using a Combination of Botulinum Toxin Type A and Hyaluronic Acid Filler: Dermatologic Surgery. 2015 Dec;41:S325-32.
19. Cartier H, Hedén P, Delmar H, Bergentz P, Skoglund C, Edwartz C, et al. Repeated Full-Face Aesthetic Combination Treatment With AbobotulinumtoxinA, Hyaluronic Acid Filler, and Skin-Boosting Hyaluronic Acid After Monotherapy With AbobotulinumtoxinA or Hyaluronic Acid Filler: Dermatologic Surgery. 2020 Apr;46(4):475-82.
20. Hedén P, Hexsel D, Cartier H, Bergentz P, Delmar H, Camozzato F, et al. Effective and Safe Repeated Full-Face Treatments With AbobotulinumtoxinA, Hyaluronic Acid Filler, and Skin Boosting Hyaluronic Acid. *J Drugs Dermatol*. 2019 Jul 1;18(7):682-9.
21. Data on file (MA-33939). Supporting data for Broadest Range Fillers.
22. Öhrlund A. Poster presented at AMWC 2019.
23. Restylane Eyelight - Instructions for Use, Galderma, December 2020.

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galdermaaesthetics.com/restylaneeyelight

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