

Radiant Skin, Radiant Patients

A progressive approach to improved skin condition

Skin quality – a common concern amongst patients

Many present with skin that has been damaged by the stresses of modern life, showing the first signs of ageing:




Loss of skin elasticity
Dehydrated skin



Deterioration of
skin structure



Appearance of fine lines
(e.g. peri-oral lines)

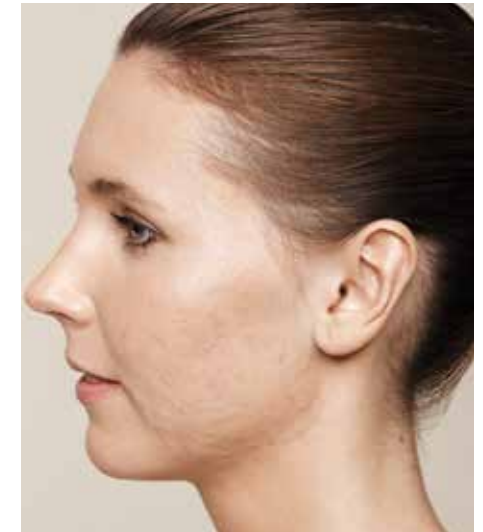

Acne scars

Most common areas of concern:

- Face
- Neck
- Hands
- Décolletage

Practitioners turn to Restylane® for a wide range of dermatological needs



Only Restylane, as the world's first non-animal stabilized Hyaluronic Acid filler, utilising the unique NASHA™ technology, provides:

- The world's broadest portfolio of fillers
- Established safety profile based on over 2 decades of experience
- Over 30 million treatments worldwide⁹



Introducing NASHA, a unique patented HA stabilization technology:

- Incorporates a limited number of synthetic crosslinks into HA structure
- Preserves natural crosslinks and entangled HA network
- Results in a minimally modified version of natural HA

Restylane® Skinboosters™: A progressive approach to improved skin condition

- Clinically proven outcomes support significant and lasting improvements to the condition of the skin ^{1, 3-8}
- A progressive treatment protocol that enables the strengthening of patient partnerships
- Visible improvements to skin condition resulting in high patient satisfaction ^{1, 3, 6}


Restylane Skinboosters provide an innovative therapy to restore skin hydrobalance, ^{2-4, 8} designed to hold water deep in the skin's dermal layer to:

- Increase skin elasticity ^{3-6, 8}
- Decrease skin roughness ^{4, 6, 8}
- Reduce fine lines ⁵
- Lessen skin imperfections, such as discrete depressed acne scars ¹



The unparalleled power of stabilized HA

Significant improvements in surface roughness

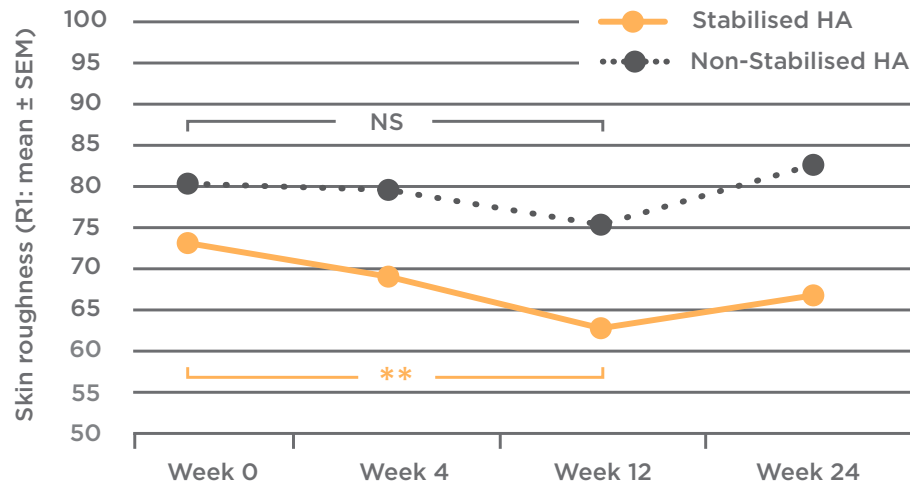

Uncovering the differences between stabilized and non-stabilized HA:

- Standard, non-stabilized HA dissipates from the injection site, limiting its effect to only a few days
- Stabilized HA gel remains in situ for much longer than non-stabilized HA
- Restylane Skinboosters build on the effects of native HA through a patented HA stabilization technique (NASHA™), delivering significant improvements in skin quality for up to 12 months^{4,5}

Restylane Skinboosters provide clinically proven improvements to skin condition at week 12 compared to treatments with non-stabilized HA⁸, including:

- Statistically significant improvements to skin elasticity and surface roughness, not observed with non-stabilized HA
- Greater improvement to clinical appearance in a blinded dermatologist assessment
- Significant improvement in clinical hand ageing scores

Clinically proven outcomes: significant and lasting improvements to the condition of the skin


Multiple clinical studies prove Restylane Skinboosters' lasting efficacy, including:

- Statistically significant improvement in skin elasticity of the face and hands observed in all patients up to 12 months after initial treatment, with sustained effect until the end of the 12 month study ^{4,5}
- Statistically significant decrease in facial skin roughness observed 6 months after initial treatment ⁶
- 100% of patients experienced GAIS scale improvement at 12 months after initial treatment ⁴

Improvements in skin quality are equally effective with needle or microcannula techniques

- No difference in GAIS, elasticity or subject satisfaction measurements up to 12 months ⁵

Skin Elasticity

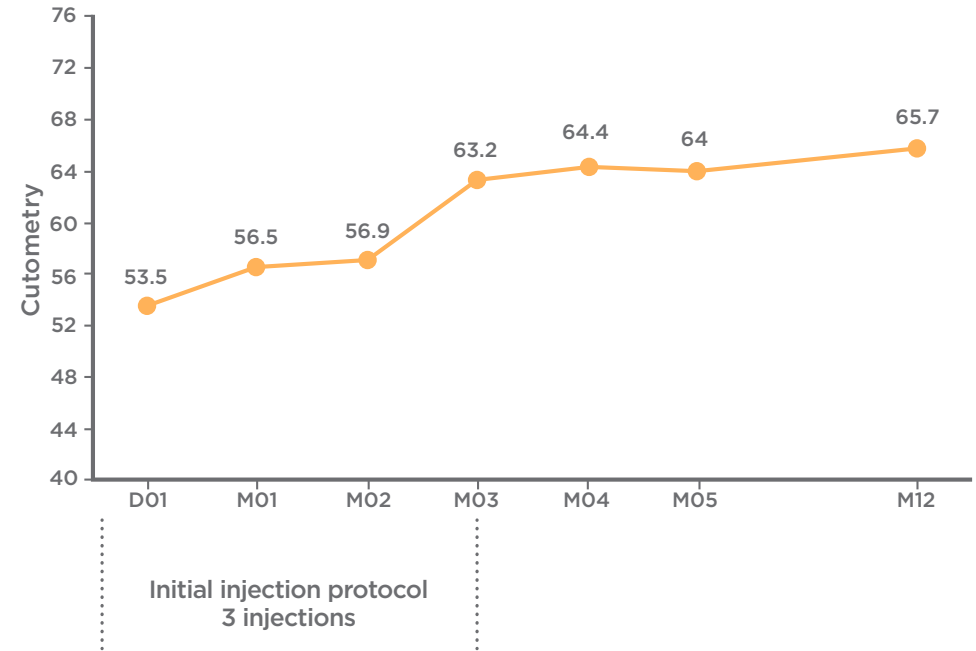

Mean elasticity values as measured by cutometry (D01-M05: n=30, M12: n=22)



A progressive treatment protocol: to enable long-term patient partnerships

Restylane Skinboosters offer the opportunity to guide your patients in a consultative partnership, helping to deliver attentive, personalized care and long-lasting results.



- Injections are typically administered over the course of 3 treatment sessions, 2-4 weeks apart
- Regular maintenance sessions take place, approximately, twice annually

Research indicates that patients find naturally radiant skin worth waiting for:

- Following treatment 100% of patients felt that their expectations were met, and this remained when asked again 6 months after the initial treatment ⁶

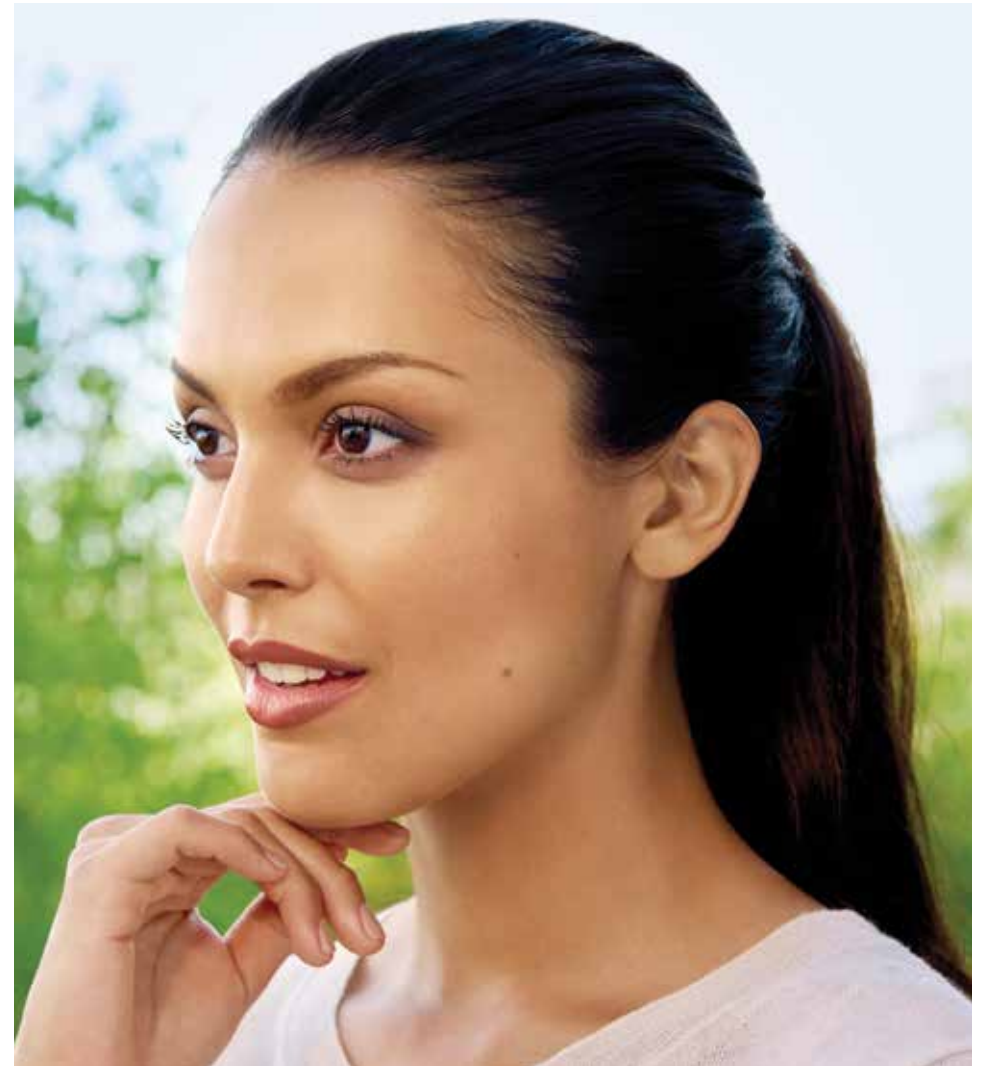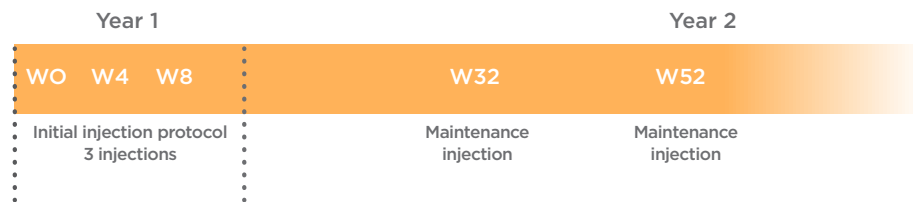


Visible improvements to skin condition result in high patient satisfaction

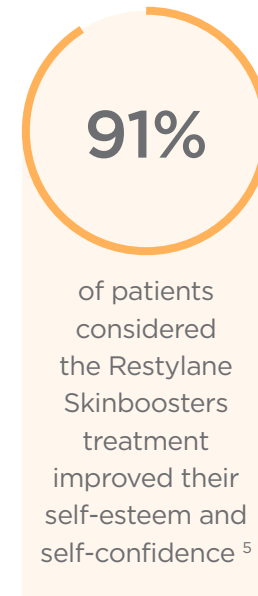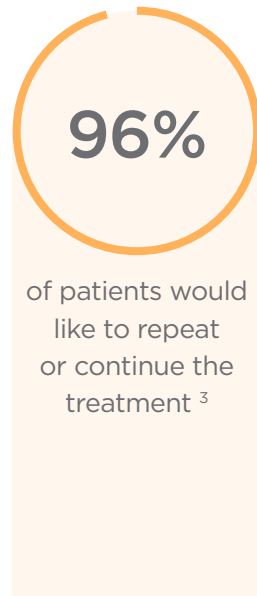

The solution for a wide range of needs

Restylane Skinboosters offer a flexible treatment option that provides skin rejuvenation and corrects a wide range of skin quality concerns, in a large number of areas.

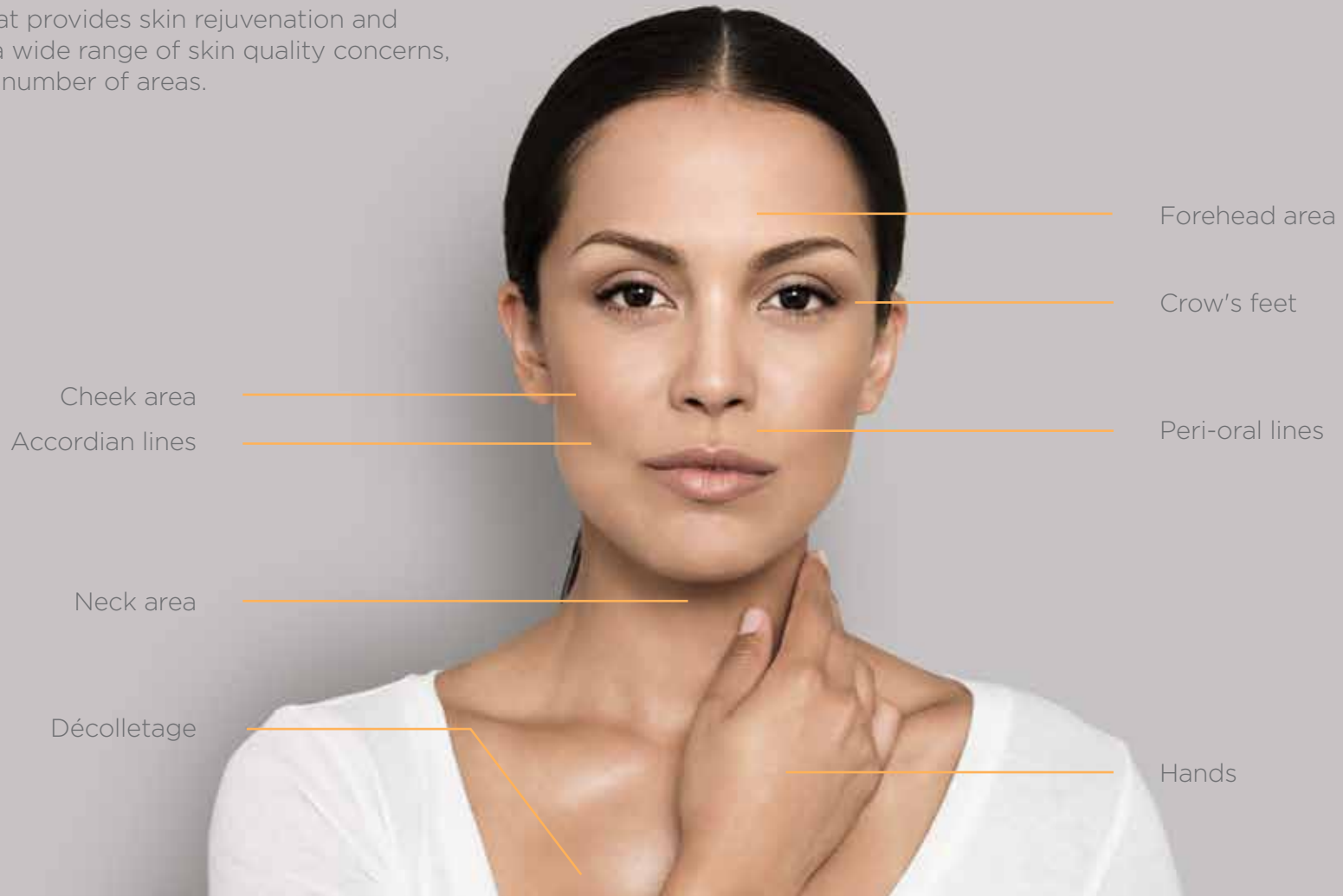

Face

Before



After



Product used: Restylane Skinboosters Vital Lidocaine

Acne scars

Before



After



Product used: Restylane Skinboosters Vital Lidocaine

Hands

Before



After



Product used: Restylane Skinboosters Vital Light Lidocaine

Décolletage

Before



After



Product used: Restylane Skinboosters Vital Light

SmartClick™:

A smarter tool for improved precision and control

Introducing the SmartClick system, designed to provide dosing consistency and pinpoint accuracy

Offers increased precision and control:

- Audible dosage indicator delivers ~10 µL microdroplets for every subtle click that you hear
- Lets you focus on the injection technique, rather than the amount injected

Provides greater working comfort:

- The SmartClick syringe feels light in the hand, with a comfortable finger grip and thumb rest ¹⁰
- Easily switch the dosage indicator on for added treatment versatility






red dot award 2014
winner

Restylane
SKINBOOSTERS™

Two formulations to enable clinical flexibility



REFRESH*	<i>Restylane Skinboosters Vital</i>	<i>Restylane Skinboosters Vital Light</i>
Composition	20 mg/mL stabilised hyaluronic acid, phosphate buffered saline, with or without 0.3% lidocaine	12 mg/mL stabilised hyaluronic acid, phosphate buffered saline, with or without 0.3% lidocaine
Technology platform	NASHA technology	
Gel particle size		
Suggested treatment areas		
Injection depth		
Treatment protocol	3 treatments, 4 weeks apart. Maintenance treatment recommended every 6 months	3 treatments, 2-4 weeks apart. Maintenance treatment recommended every 4-6 months
Onset of action	Progressive	
Packaging	1mL SmartClick™ syringe 3 x 29G TW needle	
Estimate duration	Up to 12 months ^{† 4,5}	Up to 9 months ^{† 7}

*Refreshing the look for radiant and hydrated skin

[†]Following initial treatment

Restylane Skinboosters: Right for your patients, right for your practice

- Clinically proven outcomes support significant and lasting improvements to the condition of the skin ^{1,3-8}
- A progressive treatment protocol enables the strengthening of patient partnerships
- Visible improvements to skin condition result in high patient satisfaction ^{1,3,6}



Reference List

1. Dierickx (2015). IMCAS 2015.
2. Distante et al., (2009) Dermatol Surg 35 Suppl 1, 389-393.
3. Gubanova et al., (2010) Esteticheskaya Meditsina (Aesthetic Medicine) 1, 94-98.
4. Gubanova et al., (2015a) J Drugs Dermatol 14, 288-298.
5. Gubanova et al., (2015b) IMCAS 2015.
6. Kerschler et al., (2008) Dermatol Surg 34, 720-726.
7. Streker et al., (2013). J Drugs Dermatol 12, 990-994.
8. Williams et al., (2009). J Cosmet Dermatol 8, 216-225.
9. Galderma data on file (1)
10. Galderma data on file (2)

Restylane, Restylane Skinboosters, NASHA and Galderma are trademarks owned by Nestlé Skin Health S.A.

20-22001-01

